miniland

nasal care

Español	4	Português	14	Deutsch	24	Polski	34
English	9	Français	19	Italiano	29	Русский	39

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INDEX

1. INTRODUCTION
2. CONTENTS
3. SAFETY INSTRUCTIONS
4. PRODUCT FEATURES
5. INSTRUCTIONS FOR USE
6. CLEAN INSTRUCTIONS
7. TECHNICAL SPECIFICATION
8. BATTERY & PRODUCT DISPOSAL INFORMATION

1. INTRODUCTION

Congratulations on the purchase of this ergonomic battery operated nasal aspirator. It will help you make them more comfortable; quickly and gently remove excess mucus.

Before using this device, please do read carefully all instructions in this manual.

Please keep this manual as it contains important information.

The features described in this user's guide are subject to modifications without prior notice.

2. CONTENTS

- 1 Main Unit electronic nasal aspirator
- 2 Sizes of Silicone Tips
- 2 Spare Silicone Rings
- 1 Instruction manual and guarantee card

If any of the above is missing, please contact your retailer.

3. SAFETY INSTRUCTIONS

1. This device is not a toy; do not allow children to play with it.
2. This device is to clean stuffy noses; do not use it in baby's mouth or ears.
3. Always clean and dry thoroughly before each use and after each use.
4. This is not a waterproof device, please clean it referring to the section of "Clean Instructions".
5. Do not insert the silicone tip into the nostril too deep.
6. Do not use it on the baby for a long time. A few seconds will be enough.
7. Do not use this device if there is any wound, or symptom of inflammation or bleeding at the nose.
8. In the case of infection, fever, nasal irritation or hemorrhage, please stop using this device and consult your physician.
9. This device is restricted for personal use.
10. The silicone tip is the applied part.
11. No modification of this device is allowed. You are advised that changes and modifications not carried out on the equipment by the MINILAND technical service will not be covered by the product guarantee.
12. Duty cycle: 30 seconds ON / 2 minutes OFF
13. Replace batteries when the suction power becomes weak during use.
14. This product meets the basic safety and essential performance requirements indicated in the IP22 conditioning test (IP22: Protection level against solid foreign objects and dripping water)

Battery Warnings:

Use alkaline batteries only

1. Do not mix used and new batteries.
2. Do not mix alkaline, standard (carbon zinc), or rechargeable (nickel-cadmium) batteries.
3. Only batteries of the same or equivalent type as recommended should be used.
4. Batteries should be inserted with correct polarity.
5. Exhausted batteries should be removed from the device.
6. Be sure that all contact surfaces are clean before installing batteries.
7. Risk of explosion if battery is replaced by an incorrect type.
8. Remove batteries when stored for long periods of non-use.

4. PRODUCT FEATURES

4.1. PARTS DESCRIPTION (Figure 1)

1. Silicone tips for adults
2. Silicone tips for babies and children

ENGLISH

3. Mucus container
4. Silicone rings
5. Base
6. Main body
7. Button
8. Battery compartment

5. INSTRUCTIONS FOR USE (Figure 2)

1. Remove the cover of the battery compartment (8) from the main body (6) as illustrated in the image.
2. Insert two AA 1.5V alkaline batteries (not included) to the correct poles, following the instructions printed in the compartment itself.
3. Ensure the silicone rings in the battery compartment (8) are in the correct position and close it as illustrated until you hear a "click".
4. Select the appropriate silicone tip (1, 2) in accordance with the size of the person's nostrils.
5. Ensure the silicone tip (1, 2) is in the correct position on the mucus container (3).
6. Insert the silicone tip (1, 2) in the nostril, taking care not to push it in too far.
7. Press the button (7) for a few seconds to remove the mucus. Release the button when all the mucus has been removed.
8. Repeat the operation for the other nostril if necessary.

N.B: In the event the accumulated amount of mucus is close to the maximum capacity (around 4 ml), follow the steps provided in section 6 – CLEANING INSTRUCTIONS to clean the device before using it again. Do not continue to use the device until it has been cleaned if the mucus is close to the level of the opening in the canal (illustrated in Figure 3).

6. CLEAN INSTRUCTIONS (Figure 4)

1. Ensure the battery compartment (8) is correctly closed.
2. Remove the silicone tip (1, 2).
3. Remove the mucus container (3) using the thumb and forefinger, as illustrated in the image.
4. Remove the silicone rings (4) from the nasal aspirator.
5. Wash all the parts with warm water and soap. You may place the base of the nasal aspirator (5) directly under the tap in order to wash it.
6. Reassemble all the parts as illustrated in the image.
7. Check all the parts are correctly in place and align the indicated marks.

If the amount of accumulated mucus has passed the maximum level of the mucus container (the level of the canal opening) while using the nasal aspirator, or if you notice the mucus has flowed into the area of the air outlet in the base (Figure 5):

1. Ensure the battery compartment (8) is correctly closed.
2. Remove the silicone tip (1, 2) and the mucus container (3).
3. Place the base of the nasal aspirator in clean water as illustrated in the image. Press the button (7) for 15 seconds to clean the internal air pump. The water will flow in the direction indicated in the image.
4. Change the water and repeat step 3 several times if necessary.
5. Remove the nasal aspirator from the water once you have concluded the cleaning process. Press the button (7) for around 30 seconds until the internal air pump is dry.
6. Dry the exterior of the nasal aspirator and reassemble all the parts.

7. TECHNICAL SPECIFICATION

- Operation environment:
 - Temperature: 5°C ~ 40°C
 - Humidity: 15% ~ 93%
 - Atmospheric Pressure: 700~1060 hPa
- Storage & Transportation environment:
 - Temperature: -10°C ~ 50°C
 - Humidity: ≤ 93%
 - Atmospheric Pressure: 700~1060 hPa
- Power: 2 x 1.5V AA alkaline batteries
- Shelf life: 5 years

- Suction power: 53 - 65 kPa

8. BATTERY & PRODUCT DISPOSAL INFORMATION

- Remember, when you dispose of the nasal aspirator to remove the batteries and dispose of them responsibly.
- Do not mix products bearing the crossed out bin symbol with your general household waste.
- For the correct collection and treatment of these products take them to the collection points designated by your local authority. Alternatively contact the retailer who sold you the product.

NORMALIZED SYMBOLS

- Attention: Read the operating Instructions for use!

- Applied part of type BF

- Disposal in accordance with Directive 2012/19/EU (WEEE)

- Complies with the European Medical Device Directive (93/42/EEC) and amended by directive 2007/47/EC requirements. Notified body 1639: SGS Belgium NV

- Batch Code

- Serial number

- The name and the address of the manufacturer

- Authorized Representative in the European Community

IP22

- First number 2: Protected against solid foreign objects of 12.5 mm Φ and greater Second number 2: Protected against vertically falling water drops when enclosure tilted up to 15°. Vertically falling drops shall have no harmful effects when the enclosure is tilted at any angle up to 15°, on either side of the vertical.

ELECTROMAGNETIC COMPATIBILITY (EMC)

Declaration - electromagnetic emission		
The device is intended for use in the electromagnetic environment specified below. The customer or the user of device should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.

RF emissions CISPR 11	Class B	The device is suitable for domestic establishment and in establishment directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable	

Declaration - electromagnetic immunity

The device is intended for use in the electromagnetic environment specified below. The customer or the user of device should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines	Not applicable	Not applicable
Surge IEC 61000-4-5	± 1 kV line(s) to lines ± 2 kV line(s) to earth	Not applicable	Not applicable
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT; 1/2 cycle at >95 % 40% UT; 5 cycle at 60% 70% UT; 25 cycle at 30% <5 % UT; 5s cycle at >95%	Not applicable	Not applicable
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE: UT is the a.c. mains voltage prior to application of the test level.

Declaration - electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3V 0.15 MHz to 80MHz	Not applicable	Portable and mobile RF communications equipment should be used no closer to any part of device, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1.2 \cdot \sqrt{P}$ 150KHz to 80 MHz $d = 1.2 \cdot \sqrt{P}$ 80MHz to 800 MHz $d = 2.3 \cdot \sqrt{P}$ 800MHz to 2,5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).
Radiated RF IEC 61000-4-3	3V/m 80 MHz a 2,5 GHz	10V/m	Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a Field strengths from fixed RF transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which device is used exceeds the applicable RF compliance level above, device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating device.			

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© Miniland, S.A. 2015
Miniland S.A. P.Ind. La Marjal I C/ La Patronal 10
Asia Connection Co., LTD.
Advena Ltd.