NUK®

Soft & Easy Elektrische Milchpumpe Electric Breast Pump

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MAPA GmbH
Industriestraße 21-25
27404 Zeven
Germany
www.nuk.com

Contents

1 Safety information	18
1.1 Definition of terms	18
1.2 General information	18
1.3 Hygiene	19
1.4 Information on the breast milk container and feeding teat	19
2 Important information before using for the first time	19
3 Description of device	19
3.1 Proper use	19
3.2 Models	19
3.3 Scope of supply	20
3.4 Description of function	20
3.5 Labels on the product/packaging	20
4 Key to parts (figure A)	20
5 Cleaning and sterilising	21
5.1 Cleaning	21
5.2 Sterilising	22
6 Assembling the pump	22
6.1 Selecting the silicone breast shield	22
6.2 Expressing with one breast shield	22
6.3 Expressing with two breast shields (double pump)	23
7 Power supply	23
8 Operation	23
8.1 Switching the pump on and off, selecting the mode	23
8.2 Expressing milk	24
8.3 After each pumping session	25
8.4 Feeding expressed milk	25
8.5 Storing expressed milk	25
9 Tips for successful expressing	25
10 Tips on hygiene	25
11 Storing milk	26
12 Help with problems	27
13 Disposal	28
14 Technical Data	28
15 Warranty	29
15.1 Service Centres	29
15.2 Manufacturer	29

Dear mother,

The NUK electric Breast Pump is designed as an aid for you and your baby to store up breast milk.

Please take a few minutes of your time to read these instructions for use

through carefully before using your NUK Breast Pump for the first time.

You can find brief instructions in the form of illustrations in figure D.

1 Safety information

1.1 Definition of terms

The following indicator words can be found in these instructions for use:

WARNING!

Ignoring this warning may cause serious damage to life and limb.

CAUTION!

Ignoring this warning may cause minor personal injury and/or material damage.

Note:

Issues that should be borne in mind when using the device.

1.2 General information

WARNING! The appliance is not intended to use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.

WARNING! Children must not play with the appliance.

WARNING! Keep small parts out of the reach of children: risk of swallowing and suffocation.

WARNING! Risk of strangulation. Never position tubes or cables in the vicinity of babies and children.

WARNING! Cleaning must not be carried out by children without supervision.

WARNING! The rechargeable battery is permanently built in and cannot be replaced by yourself.

An incorrectly replaced battery can result in an excessive temperature build-up, explosion and/or fire.

CAUTION! Never immerse the operating unit **17** or the USB cable **14** in water as it might be destroyed if any water gets inside.

- Please read these instructions for use through carefully before using the breast pump and keep them to hand. These instructions form part of the device and must be kept to hand at all times.
Note: Should the instructions for use become illegible or be mislaid, they can be downloaded from nuk.de or nuk.com.
- Only ever use the NUK Breast Pump and accessories for their intended purpose (see "3.1 Proper use" on page 19).
- Do not expose the device to direct sunlight as the plastic may become brittle through UV radiation and the effect of heat.
- Do not expose the appliance or battery to excessive temperatures.
- Short-circuiting the battery terminals can lead to burns or fires. This is impossible as the battery terminals are not directly accessible. The battery is charged via the USB-C port.
- We recommend removing any greasy or oily creams from your breast before using the pump as they may damage the surface of the silicone breast shield.
- Only use original NUK accessories.
- Please note that any claims under the terms of the warranty expire if

accessories that are not described in these instructions for use are utilised.

- In the event of any malfunction, the pump must be returned to the manufacturer for repair. Failure to do so will invalidate your warranty.
- Never open the housing of the operating unit **17** – **this will invalidate the warranty.**
- If the breast milk is intended for a premature baby, it is essential that you follow your doctor's advice.
- The packaging is not a toy. Keep it out of reach of children. There is a risk of injury and suffocation.

1.3 Hygiene

WARNING!

When handling breast milk, hygiene is of the utmost importance to ensure the healthy development of your child.

- Please read and pay attention to “5 Cleaning and sterilising” on page 21 to prevent infection or

bacterial contamination (microbial growth) and to prevent damage to the device.

- **The breast pump will only attain its full suction power if it is cleaned carefully.**
- **Wash your hands thoroughly before using the breast pump.**
- Please remove all body jewellery (e.g. piercings) from the breast before using the breast pump.

1.4 Information on the breast milk container and feeding teat

For your child's safety and health: ATTENTION!

Read the enclosed warnings and hints for the NUK Breast Milk Container/bottle and teat, as well as for the accessories through carefully.

2 Important information before using for the first time

WARNING! Before using the NUK Breast Pump for the first time, clean and sterilise all parts (see “5 Cleaning and sterilising” on page 21).

Charge the battery before using it for the first time (see “7 Power supply” on page 23).

Read the enclosed warnings and hints for the NUK Breast Milk Container/bottle and teat, as well as for the accessories through carefully.

3 Description of device

3.1 Proper use

The NUK electric Soft&Easy Breast Pump is an aid for expressing and collecting breast milk.

Once expressed, breast milk can be collected, stored, frozen, warmed and fed to your baby in the NUK Breast Milk Container.

The NUK Breast Pump is designed for use by a single person in the home. It is not certified for commercial use.

3.2 Models

Two models of this NUK Breast Pump are available:

- as a single pump
- as a double pump

These instructions for use describe both models.

Note: There is an extension kit (art. no. 10.252.144), which can be used to turn the single pump into a double pump.

3.3 Scope of supply

- 1 operating unit (17)
- 1 USB cable (14)
- 1 pump kit, consisting of:
 - 2 silicone breast shields of different sizes (1)
 - membrane cap (3)
 - membrane (4)
 - connecting element (5)
 - flow valve (6)
- 1 suction tube (15)
- 1 breast milk container, consisting of:
 - bottle (7)
 - screw ring (10)
 - feeding teat (11)
 - dust cap for teat (12)
 - sealing disc (13)
- 1 stand (8)
- 1 bottle adapter for NUK First Choice Bottles (9)
- 1 storage bag
- 1 set of instructions for use

The double pump additionally includes the extension kit. It contains:

- 1 pump kit, complete
- 1 breast milk container
- 1 stand
- 1 bottle adapter for NUK First Choice Bottles
- 1 suction tube set with 3 arms

Note: The extension kit (art. no. 10.252.144) for extending the single pump into a double pump can also be purchased at a later date.

Note: A USB power supply unit is not included in the scope of supply. You can use any standard USB power supply unit (mobile phone charger or power bank) with 5 V DC/min. 2 A to charge the battery. You should however always use the USB cable provided 14.

3.4 Description of function

This NUK Breast Pump is an electric suction pump offering the following benefits:

- Adjustable suction mode
- Adjustable suction strength
- Soft silicone breast shields in two different diameters
- Operated either with a power supply unit or a rechargeable battery integrated in the device.
- All materials used comply with the statutory provisions and are free from BPA.

3.5 Labels on the product/packaging

Symbol	Meaning
	Refer to the instructions for use
	Attention! Please refer to the safety information in the instructions for use
BP8	Model name
	Declaration of conformity Complies with all requirements of the applicable EC directives
	Declaration of conformity Complies with all applicable requirements for products sold within Great Britain
	Regulatory Compliance Mark (RCM) AU/NZ
	DC
	Do not dispose of the device with household waste
	Do not expose to direct sunlight
	Protect from moisture/keep dry
Art.	Article number
	Batch identification

4 Key to parts (figure A)

Pump (single or double):

- | | |
|--------------------------------------|--------------------------------|
| 1 Silicone breast shields in 2 sizes | 5 Connecting element |
| 2 Tube connection | 6 Flow valve |
| 3 Membrane cap | 7 Breast milk container/bottle |
| 4 Membrane | 8 Stand |

Bottle accessories (single or double):

- 9 Bottle adapter for NUK First Choice Bottles
- 10 Screw ring
- 11 Feeding teat
- 12 Dust cap for teat
- 13 Sealing disc

Tubes and lines:

- 14 USB cable
 - 15 Suction tube
- additionally with double pump:
- 16 3-arm tube set

17 Operating unit:

- 18 Level indicator
- 19 Time display
- 20 Mode button
- 21 Increase suction strength
- 22 Reduce suction strength
- 23 On/off button and pause/next button
- 24 Display charging state
- 25 Display drain mode
- 26 Display suction mode
- 27 Display stimulation mode

5 Cleaning and sterilising

WARNING! Hygiene is of the utmost priority when handling breast milk and all parts that come into contact with it. **The pump should therefore only be used in a clean and sterile condition.**

Wash your hands thoroughly before using the breast pump.

The following parts come into contact with your breast milk and must be cleaned and sterilised **immediately after every use** (see “5.1 Cleaning” on page 21 and “5.2 Sterilising” on page 22):

- 1 Silicone breast shields
- 4 Membrane
- 5 Connecting element
- 6 Flow valve
- 7 Breast milk container/bottle
- 9 Bottle adapter for NUK First Choice Bottles (if used)
- 10 Screw ring
- 11 Feeding teat
- 12 Dust cap for teat
- 13 Sealing disc

The following parts **must not** be cleaned and sterilised with water:

- 14 USB cable
- 17 Operating unit

5.1 Cleaning

WARNING! Milk residues may damage the material and result in microbial growth. Thorough cleaning is therefore necessary.

CAUTION!


- Clean the flow valve **6** with care so as not to damage it. We recommend only rinsing off the flow valve and boiling it. Cleaning utensils can cause damage.
- Parts should not be put in the dishwasher as detergents may damage the materials!
- No moisture should get into the suction tube **15** or the 3-arm tube set **16**. If the tube becomes moist, this may damage the operating unit. If moisture gets into the tube, it must be dried before being used again.

1. Dismantle the device as shown in figure **B**, proceeding in reverse from 7 to 1.

2. Clean the following parts with warm water and a little washing-up liquid (or NUK Rinsing Agent).

Then rinse the parts under running water:

- 1 Silicone breast shield
- 4 Membrane
- 5 Connecting element
- 6 Flow valve
- 7 Breast milk container/bottle
- 9 Bottle adapter for NUK First Choice Bottles (if used)
- 10 Screw ring
- 11 Feeding teat
- 12 Dust cap for teat
- 13 Sealing disc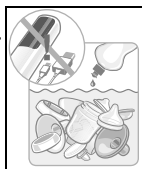

Note: When rinsed with other objects, certain foods may discolour the material (e.g. carrot juice) or affect its odour (e.g. fennel tea). This has no effect on the fitness for purpose or safety of the products.

- Then sterilise the parts (see “5.2 Sterilising” on page 22).
- Wipe the suction tube **15**, the 3-arm tube set **16** and the operating unit **17** with a damp cloth.

5.2 Sterilising

Before sterilising, always clean as described in chapter “5.1 Cleaning” on page 21.


WARNING! Risk of scalding during sterilisation.

CAUTION!

- Dismantle each part into its individual components for boiling or steam-sterilising (temperature above 80 °C) and leave open – otherwise deformation can result.
- The operating unit **17** and the USB cable **14** should never be sterilised in this way as it would cause deformation and damage the electrical contacts.

- Incorrect treatment can damage the parts.
- Before using again, always allow the sterilised parts to **cool down properly and dry completely** before reassembling.
- Do not use any chemical disinfectants for sterilising: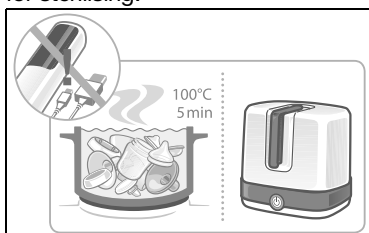

- Use a NUK Steam Steriliser, following the instructions for use of the device;

Or

- Sterilise the parts in boiling water for 5 minutes.

Ensure there is sufficient water in the pan to avoid damage.

Or

- Only use the NUK Micro Express Plus Microwave Steam Steriliser for microwave sterilisation. Follow the instructions for use of the device.


6 Assembling the pump

6.1 Selecting the silicone breast shield

The silicone breast shields **1** provided are marked on the outside as follows: 24 mm and 27 mm. This refers to the diameter at the narrowest point.


From the two silicone breast shields select the one that best suits the size of your nipple. Its diameter should be slightly larger than your nipple. Then set aside the breast shield that is not the right size to rule out any confusion.

6.2 Expressing with one breast shield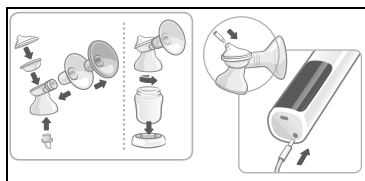

- Assemble the device as shown in figure **B**, steps 1 to 7.
- If you are expressing with only one breast shield but are using the three-arm tube set **16** for the double pump, you must insert the unused tube in

the cross-piece to prevent any air being drawn in there.

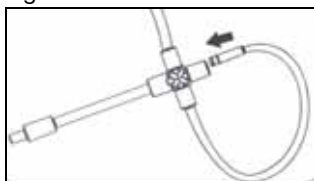Note: The bottle adapter **9** provided also allows you to use the breast pump with bottles from the NUK First Choice range.

Note: Make sure that the silicone breast shield **1** fits snugly with the connecting element **5** and that the flow valve **6** is

mounted properly as the pump may otherwise draw in air, so preventing it from working correctly.

6.3 Expressing with two breast shields (double pump)

1. Assemble the extension kit as described in “6.2 Expressing with one breast shield” on page 22.
2. Attach both tube connections **2** to the operating unit **17** using the three-arm tube set **16** (see figure **C**).

7 Power supply

The operating unit **17** is equipped with a permanently installed rechargeable lithium battery with a capacity of 1400 mAh. The capacity of the fully charged battery is sufficient for around 3 to 4 expressing sessions.

The charging state of the battery is indicated on the operating unit by the display **24**.

To charge the battery, connect the operating unit **17** to a USB power supply unit (mobile charging device, power bank) with 5 V DC / 2 A using the USB charging cable **14** provided.

As soon as the battery is charging, this is indicated by bars running in the charging display **24**.

Note: The battery can also be charged while expressing.

8 Operation

Note: During operation it is important to only touch the button required and as vertically as possible. For safety reasons, nothing will happen if you accidentally press two buttons at the same time.

8.1 Switching the pump on and off, selecting the mode

1. Touch the on/off button **23** for at least 2 seconds until the display on the operating unit **17** lights up.

The pump is now in pause mode. The display **18** is lit up.

2. Touch the mode button **M 20** to change the mode.

Selecting the mode:

	Stimulation mode	Suction mode	Drain mode
Description	In stimulation mode your breasts are gently stimulated to encourage the flow of breast milk. Your milk will start to flow. The device will automatically switch to suction mode after 2 minutes.	The pump works with long suction cycles to drain milk from the breast. The let-down reflex is triggered and your breast milk will flow.	At the end of the expressing process drain mode drains all milk from the breast. The pumping action is 3x short and 1x long. Further milk production may be encouraged in this mode.
Comparison with breast-feeding	Simulation of the initially fast feeding phase of babies from the breast when satisfying their thirst. Duration 2 minutes.	Simulation of the second feeding phase of babies. Drinking more calmly in longer cycles.	When the flow of milk decreases, babies feed faster again to stimulate the flow of milk once again.
Symbol in display			
Number of levels	9	9	9
Select level	Using the 21 + and 22 - buttons		

3. Touch the on/off button **23** again to stop the pump and switch to pause mode.

To switch the pump off altogether, touch the on/off button **23** for at least 2 seconds.

Automatic switch-off

If you do not use the breast pump for more than 30 minutes, it will switch off automatically.

8.2 Expressing milk

WARNING! As hygiene is particularly important, please wash your hands thoroughly before expressing milk.

8.2.1 Using a single pump

1. Position the silicone breast shield **1** on your breast so that your nipple is in the middle. Gently press the silicone breast shield in place to ensure a good seal for expressing your milk. **When doing so, keep the**

breast milk container 7 attached to the silicone breast shield in an upright position.

2. Touch the on/off button **23** again.

The device starts in stimulation mode, with the suction strength at level 1.

The suction strength is shown on the display **18**.

Caution: Begin with suction strength level 1 and then increase as required. We recommend going down a level if expressing milk is painful.

3. Use the **21 +** and **22 -** buttons to select an intensity that you find agreeable from the 9 levels.

The suction strength selected is shown in the level indicator **18**.

4. If required, increase the suction strength with the plus button **21** or reduce it with the minus button **22**.

The device will automatically switch to suction mode after 2 minutes.

The suction strength selected is shown in the level indicator **18**.

5. If required, increase the suction strength with the plus button **21** or reduce it with the minus button **22**.

The suction strength selected is shown in the level indicator **18**.

6. If you notice that your flow of milk is decreasing, touch the **M 20** mode button to switch to drain mode (see “8.1 Switching the pump on and off, selecting the mode” on page 23).
7. Use the **21 +** and **22 -** buttons to select an intensity that you find agreeable from the 9 levels.

The suction strength selected is shown in the level indicator **18**.

8.2.2 Using a double pump

1. Proceed as described in the previous section for the single pump.
2. However switch the pump on first.
3. Only then should you position both silicone breast shields **1** on your breasts simultaneously.

Note: If you wish to have both hands free, place the silicone breast shields in a special bra designed for expressing milk. Such bras have an opening at the nipple where breast shields can be secured. Make sure that the bra adequately presses the breast shields onto the breast so that the pump does not draw in any air and will then fail to express properly.

8.3 After each pumping session

1. Switch the pump off by touching the on/off button **23** for at least 2 seconds until the display goes out.
 2. Place the pump with the attached breast milk container **7** in the stand **8**.
 3. Pull the tube **15** from the tube connection **2**.
 4. Unscrew the connecting element **5** with the breast shield **1** and membrane cap **2** from the breast milk container **7**.
- Make sure you hold the breast milk container upright while doing so.**
5. After completing your pumping session immediately clean the individual components, except for the filled breast milk container (see “5 Cleaning and sterilising” on page 21).

8.4 Feeding expressed milk

Screw the feeding teat **11** onto the breast milk container **7** with the screw ring **10**.

8.5 Storing expressed milk

If the breast milk you collected is not for immediate use, seal the filled breast milk container firmly **7** with the sealing disc **13** and screw ring **10** and either store in the refrigerator or freeze it (see “11 Storing milk” on page 26).

9 Tips for successful expressing

Expressing correctly and without discomfort takes practice. The expressing process may vary depending on how full the breast is and its form on each day. Try out the different modes and see what suits you best. You will initially need to get used to your new pump.

You can find information on successful expressing and many other handy tips in our NUK guide “Breastfeeding”, which is available online at:

https://www.nuk.co.uk/en_gb/help-advice/downloads/nuk-little-books

10 Tips on hygiene


Hygiene is particularly important when using the breast pump. This applies to all the individual pump components as well as to your personal hygiene. We recommend regularly washing your

nipples in clean water. Where possible, avoid using soap on your nipples as this can cause irritation.

11 Storing milk

11.1 In the breast milk container

Breast milk may be chilled, frozen and defrosted in the breast milk container **7**. The breast milk container **7** may also be used for warming and feeding. It is important to note the information about handling breast milk given below in “11.2 Important information for handling breast milk” on page 26.

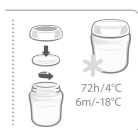WARNING! For hygiene reasons, please note:

If breast milk is not used immediately, you must seal the filled breast milk container firmly and store it in the refrigerator or freeze it.

Use a suitable pen to write the date and time you expressed your milk on the breast milk container or use labels. Make sure you observe the storage periods in “11.2 Important information for handling breast milk” on page 26.

CAUTION! To avoid any damage:

- Do not place the entire breast pump in the refrigerator, just the breast milk container **7** with the screw ring **10** and sealing disc **13**.
- When freezing breast milk, do not fill the container with more than 150 ml (5 fl.oz), because breast milk expands as it freezes and the container might burst.

The NUK Storage Bag for breast milk also offers a space-saving solution for freezing and storing your precious breast milk.

11.2 Important information for handling breast milk

Breast milk	Storage location	Storage periods
Freshly expressed	At room temperature	Max. 3-4 hours
	Refrigerator (not in the door)	72 hours at 4 °C
	Refrigerator freezer compartment	2 weeks
	Upright/chest freezer	6 months at -18 °C
Defrosted, unopened	Refrigerator (not in the door)	24 hours at 4 °C
Defrosted, opened	Refrigerator (not in the door)	12 hours at 4 °C
Defrosting milk	Gradually in refrigerator	Approx. 24 hours at 4 °C
	At room temperature followed by immediate feeding	
	If you are in a hurry: Defrost the container of breast milk under running water, cold or luke-warm (max. 37 °C), then use straight away.	

WARNING! To ensure adequate hygiene at all times: If transporting chilled breast milk make sure that it stays chilled at all times, or the milk could go off and be harmful to your baby. Use a cooler bag with ice packs, for example.

Breast milk must not be defrosted in a microwave as this will destroy valuable nutrients as well as causing possible damage to the material of the container, with a possible risk of explosion. Uneven heating may result in scalding!

Defrosted, reheated or unused milk must be disposed of and not refrozen.

1. To attain the correct feeding temperature of approx. 37 °C, place the container of defrosted breast milk in a NUK Baby Food Warmer.

CAUTION! The container must be open to prevent a build-up of pressure as the milk is warmed.

2. Before feeding, gently shake the milk so the milk and the fat in it are mixed well again.
3. If you express breast milk more than once within a period of 24 hours, the freshly expressed milk may be added to already cooled breast milk under the following conditions:

The temperatures must be the same (cold on cold). Freshly expressed milk must therefore be cooled first before being added to milk that is already cool.

Notes:

- Consult your lactation consultant, midwife, doctor or chemist for more detailed information about long-term storage.

12 Help with problems

Problem	Cause	Remedy
Pump does not start	On/off button 23 was not pressed correctly	Touch the middle of the on/off button 23 from above for at least 2 seconds. No response from the device when several buttons are touched simultaneously
	No power supply	If the battery is flat, connect the USB cable (see “7 Power supply”)
Suction pressure weak	Tube kinked	Check tube and remove kinks
No suction pressure	Tube not positioned correctly on the tube connection	Fit the tube onto the tube connection again
	When using the three-arm tube set with only one bottle, a connection is open	Close second tube access (see “6.2 Expressing with one breast shield”)
	Flow valve 6 is missing, fitted incorrectly or damaged	Check for correct positioning. With damage, use the spare valve provided
	Parts not assembled correctly	Check for correct assembly (see “6 Assembling the pump”)
Pump operating but not drawing in milk	Silicone breast shield 1 is not attached to the connecting element 5 correctly (leakage)	Make sure that there is an airtight connection between the silicone breast shield and the connecting element
	No airtight connection between the silicone breast shield 1 and the breast	Position the silicone breast shield on the breast so it is airtight (see “8.2 Expressing milk”)

Problem	Cause	Remedy
Insufficient pump suction	Suction strength is set too low	Increase the suction strength by touching the plus button 21 (see “8.2 Expressing milk”)
Pump suction too powerful	Suction strength is set too high	Reduce the suction strength by touching the minus button 22 (see “8.2 Expressing milk”)
Battery is not charging	Incorrect USB cable is in use	Only use the USB cable provided
Spare parts are needed		Contact Service Centre (see “Contact addresses” on page 142)

13 Disposal

All parts can be disposed of with your household waste, apart from the operating unit and the USB cable.

The operating unit and the USB cable must be taken to a collection point for recycling electrical and electronic equipment.

14 Technical Data

Power supply – Inlet	USB Type C compatible port (no further USB function; power supply only) 5 V DC / 2 A
Rechargeable battery – Capacity – Replacing – Leakage of liquid – Disposal – Battery operating time – Expected service life	Li-ion 3.7 V; 1400 mAh The battery is built into the device and cannot be replaced. If liquid leaks from the device, do not touch the device or only touch it with gloves and dispose of the device properly. See “13 Disposal” on page 28 If fully charged approx. 1.5 hours 400 h
Environmental conditions – Atmospheric pressure – relativ humidity – Temperature operation – Temperature storage/transport	70 kPa – 106 kPa 15% - 75% +5 °C to +30 °C -10 °C to +50 °C

15 Warranty

Subject to careful treatment of your appliance and observance of these instructions for use, the manufacturer offers a 24 month warranty from the date you purchase the product.

Appliances sent to us postage paid will be repaired free of charge in the event of faulty materials or workmanship. Any further claims are in accordance with your statutory rights.

No compensation will be given for damage caused by improper use of the device.

Wear of the cable or plug is excluded from the warranty.

Within the first 24 months after the date of purchase we will rectify any recognised faults without any charge for incidental costs incurred.

Under all circumstances, the device must be returned accompanied by this completed warranty certificate or, failing this, the purchase receipt or other proof of the purchase date must be supplied.

If no proof of purchase date is provided, you will be invoiced for the costs incurred without further inquiry, and the item will be returned to you with a request of payment on delivery.

Please complete the warranty form immediately after purchase and keep this, or the till receipt, in a safe place as proof of purchase. Lost warranty certificates will not be replaced.

Please keep the address, Art. No. and LOT No. for future reference.

15.1 Service Centres

The addresses for service can be found under "Contact addresses" on page 142.

15.2 Manufacturer

Please note that the address below **is not a service address**. In the event of problems or queries relating to your product, contact one of the Service Centres "Contact addresses" on page 142.

MAPA GmbH
Industriestraße 21-25
27404 Zeven
Germany
www.nuk.com

Warranty only valid in Australia and New Zealand

This NUK product is covered by a 24 month replacement or repair warranty, which is in addition to your rights under

the Australian Consumer Law (if your product was purchased in Australia) or New Zealand Consumer Guarantees Act (if your product was purchased in New Zealand).

Upon receipt of your claim, NUK will seek to resolve your difficulties or, if the product is defective, advise you on how to obtain a replacement or refund.

In order to make a claim under our warranty, you must have the original proof of purchase documentation for the product and present it when requested.

Should your product develop any defect within 24 months of purchase because of faulty materials or workmanship, we will replace or repair it, at our discretion, free of charge. A product presented for repair may be replaced by a refurbished product of the same type rather than being repaired. Refurbished parts may be used to repair the product.

Our replacement or repair warranty only applies where a defect arises as a result of faulty material or workmanship during the warranty period. Your warranty does not cover misuse or negligent handling, accidental damage, or normal wear and tear.

Your warranty does not:

- cover freight or any other costs incurred in making a claim;
- cover damage caused by: power surges, power dips, voltage supply problems, or use of the product on incorrect voltage;
- servicing or modification of the product other than by NUK;
- use of the product with other accessories, attachments, product supplies, parts or devices that do not conform to NUK specifications;

or

- exposure of the part to abnormally corrosive conditions;

or

- extend beyond 3 months if the product is used in commercial, industrial, educational or rental applications.

The benefits given to you by our warranty are in addition to other rights and remedies under law in relation to the product.

In Australia our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

Our goods also come with guarantees that cannot be excluded under the New Zealand Consumer Guarantees Act.

If your warranty claim is not accepted, we will inform you and if requested to do so by you, repair the product provided you pay the usual charges for such repair. You will also be responsible for all freight and other costs.

Should your product require repair or service after the warranty period, contact your local NUK customer service centre.

For more information, visit our website or call our customer care line for advice on:

1800 804 918 in Australia,

or

0800 333 006 in New Zealand.

Should you experience any difficulties with your product during the warranty period, please contact NUK Customer Care.

For distributor addresses please refer to page 142.

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AU - AUSTRALIA

Newell Australia Pty Ltd
Caribbean Park
Level 3, 35 Dalmore Drive
Scoresby VIC 3179
www.nuk.com.au

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GB - GREAT BRITAIN

Newell Brands UK Limited
Halifax Avenue, Fradley Park
Lichfield, Staffordshire
WS13 8SS
United Kingdom
www.nuk.co.uk

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☎ +36-1-248-3030
office@maresi.hu

IE - IRELAND

Intrapharma Consumer Health
Magna Business Park
Citywest Road
Dublin 24
United Drug House
☎ +353 1 463 2424

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LT - LITHUANIA

German Products Lithuania UAB
Rugiu g. 12-17
Vilnius, LT- 08430
Lithuania
info@gpbaltics.lv

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TİCARET VE SANAYİ LTD ŞTİ
Artemis Brands (PTY) LTD